



TINEARITY™ G1 FOR PROFESSIONALS

Tinearity™ G1

The Tinearity G1 is a tinnitus masking and treatment device designed to allow users to relieve and treat tinnitus - anytime, anywhere. The United States Food and Drug Administration cleared the Tinearity G1 for marketing and sales in the United States in mid-2023.

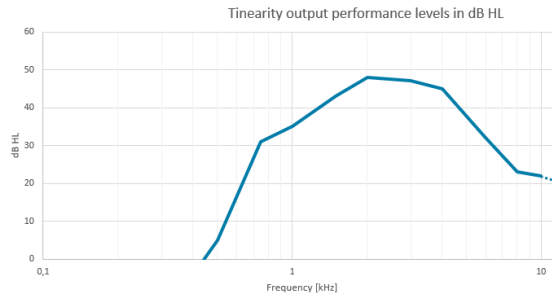

Device data	Tinearity™ G1
MFO	48 dB HL
Peak frequency	2 kHz
Battery time	min 8 hours

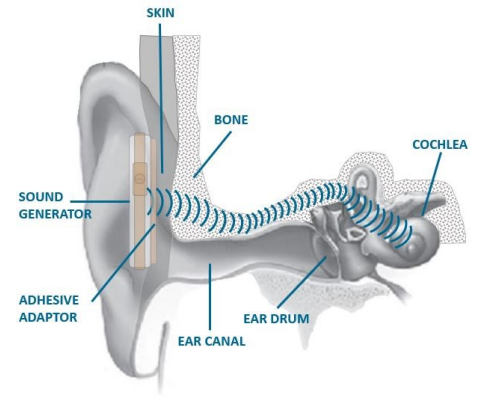

HOW IT WORKS:

Sound generators snap into adapters adhered to the skin behind each ear. White noise is transferred directly into the cochlea by bone conduction to mask or treat tinnitus. Sound vibrations are transferred by bone directly to the inner ear bypassing the external ear and middle ear structures. This unique design provides several advantages to include:

- Normal speech is not obstructed in the ear canal.
- You can sleep with the device without irritating the ear canal.
- The noise generators can reside discreetly behind the user's hair.

TREATMENT:

The Tinearity G1 can enhance CBT through use of wearable white noise.

US clinicians have recommended wearing the Tinearity G1 device for 6-8 hours per day for 6-8 months of treatment.

Use our Find a Clinic tool to make an appointment and determine if the G1 is the right device for your tinnitus symptoms.



Daily Use Adapters

Tested for irritation and allergic reactions



Two Sound Generators

White noise treatment provided via bone transfer



One Charging Plate

Sound generators are charged wirelessly

Version 3.0



info@duerarity.com



www.duerarity.com



DUEARITY
THE TINNITUS COMPANY